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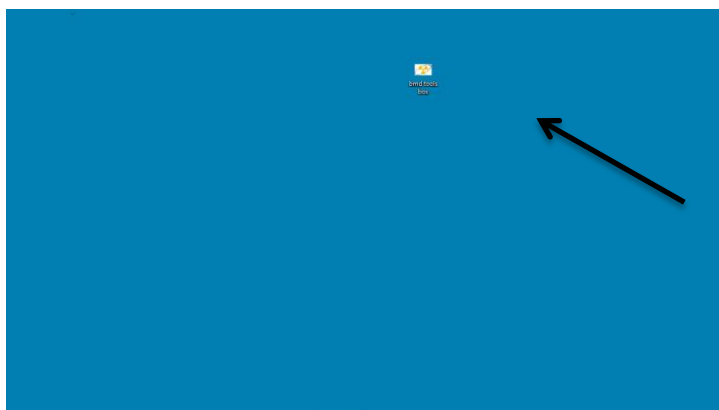
www.sinaparto.com/normal

Most Visited Suggested Sites http://sinaparto.com/... Getting Started مطالعه تپهتری Web Slice Gallery

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NEDA IMAGING CLINIC

Device model: norland
serial number: xr600
Request doctor: dr. borhanazad
Patient name: mahyar
sex: male
address: caminabad

The Bone Mineral Density (BMD) of AP Spine was measured on above person, using Dual Energy X-ray Absorptiometry (DEXA) method by norland., xr600 densitometer.

According to the recommendation of the International Dual X-Ray Absorptiometry (DXA) Standardization Committee (ISDC), the results of BMD will be reported as Standardized BMD (sBMD).

SUMMARY OF RESULTS

WHO DEFINITION	age	T-SCORE	REGION
Normal	20	-1	L1-L4 AP SPINE

For prevention of osteoporosis in this person, the following is recommended:

1. Avoid smoking, excess coffee and alcohol use.
2. Regular exercises.
3. Calcium intake of 1200-1500 mg/day in form of dietary intake + supplement if needed.
4. Vitamin D, 400-800 units/day.

SINCERELY YOURS
CONSULTANT RADIOLOGIST DR. FOULADY

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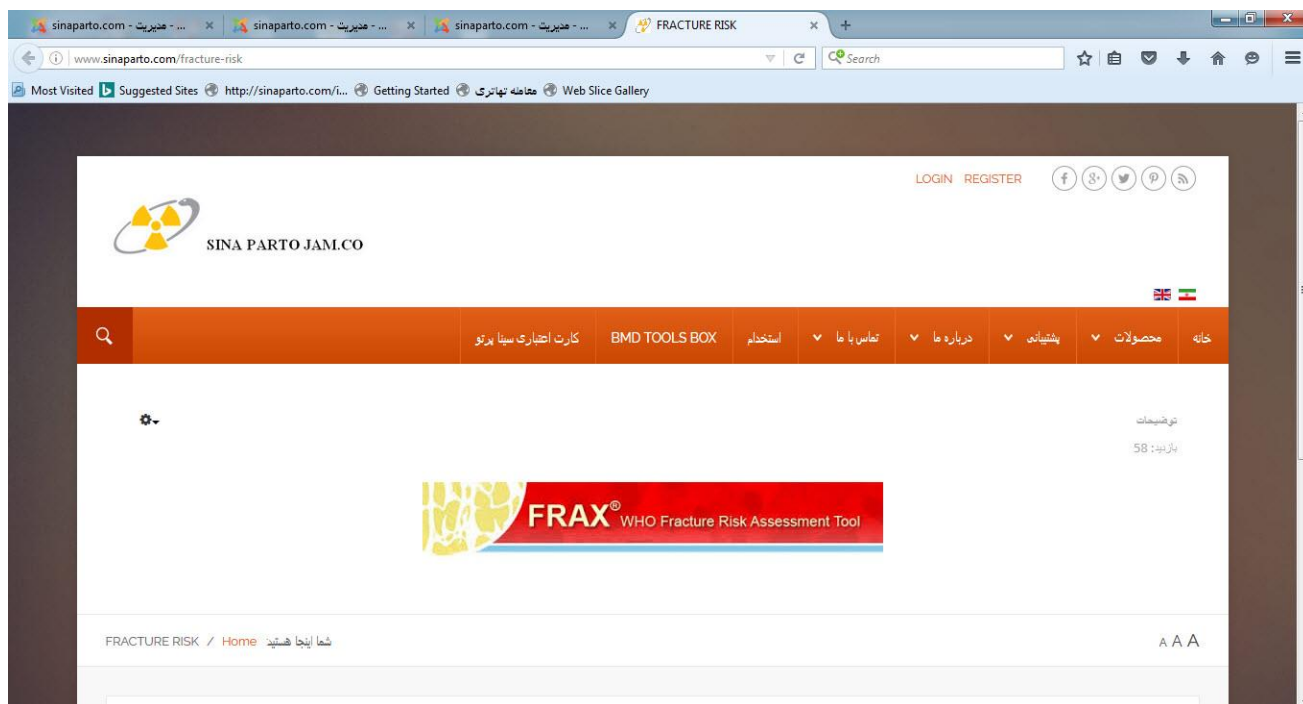
A screenshot of the 'FRAX WHO Fracture Risk Assessment Tool' Calculation Tool interface. The page has a red header with the tool's name and a navigation menu. The main content area is titled 'Calculation Tool' and contains a questionnaire for calculating the ten-year probability of fracture. The questionnaire includes fields for Country (UK), Name/ID, Age, Date of Birth, Sex, Weight (kg), Height (cm), Previous Fracture, Parent Fractured Hip, Current Smoking, Glucocorticoids, Rheumatoid arthritis, Secondary osteoporosis, Alcohol consumption, and Femoral neck BMD. There are also sections for Weight Conversion (Pounds to kg) and Height Conversion (Inches to cm). A 'Calculate' button is visible at the bottom right of the questionnaire. The footer shows the URL 'http://www.shef.ac.uk/FRAX/tool.jsp?locationValue=9' and a reference number '04231607'.

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sinaparto.com

sinaparto.com/index.php?option=com_rsform&formId=41

Most Visited Suggested Sites http://sinaparto.com/i... Getting Started مشاهده تهاثری Web Slice Gallery

NEDA IMAGING CLINIC

Patient name: BORHAN

sex: male

age: 30

BMI (Body Mass Index) is a measurement of body fat based on height and weight that applies to both men and women between the ages of 18 and 65 years. BMI can be used to indicate if you are overweight, obese, underweight or normal. A healthy BMI score is between 20 and 25. A score below 20 indicates that you may be underweight; a value above 25 indicates that you may be overweight.

Patient bmi: 31.02

BMI	Women	Status
< 20.7	< 19.1	Underweight
20.7 - 24.4	19.1 - 25.8	Normal
24.4 - 27.8	25.8 - 27.3	Marginally overweight
27.8 - 31.1	27.3 - 32.3	Overweight
> 31.1	> 32.3	Obese

Sincerely Yours
CONSULTANT RADIOLOGIST DR. FOULADY

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